



Dash Sleek Rollator User Manual



Manufacturer Company:

Address:

After-Sales Service Contact Number;

E-mail Address:

Before operating rollator walker,
Please read the user operating manual and keep it for future reference.

·INTRODUCTION

The rollators are manufactured from high-quality aluminum alloy materials and include a seat,bag,two armrests,adjustable handle, Two rear leg tubes equipped with wheels and locking brakes.The hand-bar are adjustable in height and the rollator folds easily for transport or storage.

·INTENDED USE

Designed to assist the user to walk,rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown.The intended user should possess the necessary physical,visual,and cognitive skills to operate the rollator safely.If you have any doubt,please contact qualified technician.It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

·SAFETY ADVICE

This rollator is designed to be used as a mobility aid,creating a natural gate as the user walks securely with their primary weight between the handle bars with the unit in front of the user.Read this manual carefully before using your rollator.

·DO

- (1)Always ensure the brakes are in the locked position if the rollator is left unattended.
- (2)Check all castors and wheels are secure before use.

- (3)Check the brakes are functioning properly.
- (4)Check handle height adjustment hand Button are secure before use.
- (5)Check the rollator is fully unfolded.
- (6)Take care when carrying the rollator when folded.
- (7)Keep your hands away from the folding mechanism to prevent any risk of pinching.

·DO NOT

- (1)Sit on the rollator if the brakes are not locked,this could cause the rollator to move resulting in injury to the user or other persons.
 - (2)Drop into the seat of the rollator,always sit into the rollator in a smooth,controlled motion.Seek assistance if necessary.
 - (3)Hanging objects on the brake levers or hand-grips,this could affect brake operation and/or stability of the rollator.
- Every care has been taken to ensure that your rollator will provide safe and reliable service as you adhere to the advice given above.

·LIST OF COMPONENTS

- 1.Shopping Bag
- 2.Main frame with Two armrests
- 3.Backrest

·SPECIFICATION

Overall width 62.2cm

Overall length 64.4cm

Seat height 55cm

Adjustable handle height 81.2-91.4cm

Max.user weight 150kg

Net weight 6.4kg

Adjusting the height

Release the latch buckle, press the button on the height adjustment on the frame of the rollator and adjust the handle to the appropriate height.

Repeat for the other side.Make sure the buckle is engaged , the buttons on both handles are locked.Make sure that both handles are set to the same height.

Handle height 81.2-91.4cm

Your ideal handle height

Adjust the handles of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle,your elbow should be slightly bent.If you find that you need to bend over or keep your arms bent at a right angle,adjust the height of the handles.



Brake



1.Pull the brake handles up to brake the rollator.



2.Push the brake handles down to lock the rear wheels.In this, position,the parking brake of the, rollator is active.To release the parking brake again,pull the brake handles upward.

3.Brakes must be applied in the lock position when the seat is going to,be used.Precheck the lock by pushing forward on the rollator.

Folding and storing the rollator



1.Pull the handle in the middle of the seat firmly upwards to fold the rollator

·MAINTENANCE

Over a period of time,the brake cables may stretch,and if no further adjustments can be made,the cables should be replaced.Seek the help of the manufacture when replacing brake cables.The castors should be checked that they are secure each time before using the rollator.Also,check the brakes operate correctly and the handles and hand knobs are secure.Should any part of the rollator appear loose or insecure,discontinue use until it is inspected by a qualified technician.The following shows the maintenance schedule:

Weekly

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs

Periodically

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tires for wear or cracks

Yearly

- Once a year have your rollator fully inspected,cleaned,maintained and lubricated with all moving parts

Serviceable Components

- Wheels
- Hand grips
- Bag
- Knobs
- Cane holder
- Standing armrest sleeve
- Armrests
- Backrest

· CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water.Domestic anti-bacterial surface cleaners may be used if required.Care should be taken not to leave any cleaning residues on the hand grips or seat(where fitted)which may leave them slippery when used.Do not use a scouring pad for scrapers this may damage surface coatings.

·WARNINGS!

- 1.Maximum working limit 150kg.
- 2.Do NOT stand on this product.
- 3.Ensure both push handles are adjusted to the same height.
- 4.Seek advice from a trained healthcare professional for height adjustment of the product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.

6. Some components may cause harm or damage when they come into contact with other objects.

7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require a replacement, please contact a qualified technician.

8. Ensure that your rollator is opened fully.

9. Ensure that your fingers are clear from the scissor arm when folding.